## Protac SensCircle<sup>®</sup>

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GB - Directions for use

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SE - Bruksanvisning

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Protac SensCircle®

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# GB - Directions for use

Protac SensCircle®

You have chosen a Protac quality product. Protac guarantees that you have acquired an original, Danish design product, developed in cooperation with professional business partners. Since 1994, Protac has been designing, producing and selling sensory-stimulating devices for the health sector in Denmark, Europe, Australia and New Zealand.

Protac's products are developed on the basis of theories about sensory integration and are well-evidenced in terms of research, documentation and case studies. In the manufacturing process, Protac focuses on environmental and safety issues. Protac's plastic balls are TÜV-approved, and both the polyester and flame-retardant fabrics are Oeko-Tex® certified.

GB

## Labelling

Products are labelled in compliance with regulations for medical equipment. A label with information is attached inside the cover to protect the user from discomfort, when s/he is using the pad. The position also protects the label against wear and tear.

A label with information about the manufacturer, basic components and washing instructions is attached to the individual parts:

- Outer cover
- Modules with 50-mm plastic balls
- Modules with granules (polystyrene)

Key to label symbols: please refer to **Key to symbols**

## General precautions - Safety

- ⚠ • **Pay attention to** the user's reaction if s/he does not like being encased by the pillow. This might be expressed by bursts of anger or anxiety.
- ⚠ • **Make sure that** the user does not feel too hot, when encased by the pillow.
- ⚠ • **Make sure you** use Protac SensCircle® appropriately to avoid the danger of tripping up.
- ⚠ • **Be aware of** whether any balls are causing the user discomfort. This is particularly important if the user is unable to move.
- ⚠ • **It must never be used, without supervision**, for users who have breathing difficulties.
- ⚠ • **It must never be used, without supervision**, for users who cannot take care of themselves, either physically or mentally.
- ⚠ • **Do not** use outdoors.
- ⚠ • **Do not** use near an open flame.

## The product's purpose and application environment

Sensory integration is the basis for the design of all Protac's products. This theory originated in the United States and was developed by A. Jean Ayres, an occupational therapist and psychologist, back in the 1970s. The theory is based on how the brain processes sensory impressions, and how one can influence the various sensory stimuli via weight and tactile sense (the sense of touch) in the skin and the proprioceptive sense (sense of movement) in the muscles and joints. The pressure, movement and weight of the balls have an impact on these senses.

**Protac SensCircle®** is designed as an aid for people with disabilities. Adults, youngsters and children can use **Protac SensCircle®**:

- Psychiatric illnesses such as schizophrenia, depression, anxiety, anorexia etc.
- Children and youngsters with sensory disturbance
- ADHD and autism
- Dementia problems, Alzheimers disease
- Brain damage, including Huntingtons disease, Parkinsons disease etc.
- Developmental irregularities, deafness and blindness
- Stress

Protac SensCircle® can be used for individuals suffering from psychiatric illness, who often experience motoric and mental unrest. Protac SensCircle® provides a sense of security. It encases and demarcates the body, making the user feel safe and calm.

GB

Protac SensCircle® can be used for elderly people in care homes, who may suffer from dementia.

Protac SensCircle® provides a sense of security and can be positioned so that the user can lie down and relax.

Protac SensCircle® can be used for the sensory stimulation of individuals with brain damage.

Protac SensCircle® can be used for individuals with ADHD and autism, encasing them with the balls and providing them with the sensory stimuli necessary to create a sense of security and calm.

Protac SensCircle® can be used for individuals with the likes of developmental irregularities, sensory disturbance and deafness and blindness to enhance sensory stimuli for the body and improve users' body-awareness.

Protac SensCircle® can be used for individuals with stress to give them calm, a sense of security and enhanced body-awareness.

Protac SensCircle® can be used to promote a good night's sleep. It helps calm down the central nervous system, because the user gets a sense of him/herself.

Protac SensCircle® can be used in hospitals, care homes, institutions, residential facilities, day-care centres, common rooms and private rooms.

## Use of the product

Protac SensCircle® consists of 3 modules and an outer cover. Each individual module is filled with 50-mm plastic balls or granules (polystyrene).

The uses of Protac SensCircle® include:

- Sitting up as a back support, for example in bed
- A personalised reclining position
- Packing the body
- Demarcating other other parts of the body
- A chair
- A cave
- Lying on

GB

Protac SensCircle® comes ready assembled.

## Taking apart prior to washing:

1. Zip open the outer cover and remove the 3 modules.
2. Close the zip in the outer cover prior to washing.

## Putting together after washing:

1. Zip open the outer cover.
2. Insert the 3 modules.
3. Zip up the outer cover.

GB

## Accessories

- Outer cover
- Modules with 50-mm plastic balls
- Modules with granules (polystyrene)

## The product's service life

The product has an expected service life of 3 years, though the granule (polystyrene) modules have a service life of 2 years.

## Cleaning

The service life is based on proper use, cleaning, washing, service and maintenance. Products, which are used for longer than the above-mentioned service life, should be assessed prior to further use. It is the responsibility of the user to check regularly whether the product can be used for longer than the stipulated service life. Both the outer cover for the Protac SensCircle® and the covers of the individual modules containing the balls are washable. The covers of the modules can be wiped with spirit. How frequently it needs washing depends on use. For washing and cleaning the Protac SensCircle®, please follow the instructions below.

- The outer cover can be washed in a washing machine with a min. capacity of 6 kg, using washing powder without bleach. Also see the washing instructions in the cover.
- The modules can be washed separately in a washing machine in the wash bags supplied. Min. 16 kg using washing powder without bleach. Also see the washing instructions in the individual modules.
- The covers of the modules can be wiped with spirit.

For removing stains, when you are considering using a stain remover, first try it on a place that is not visible, to see whether it might damage the fabric or affect the colour. Protac does not recommend the use of stain removers.

Protac reserves the right not to replace a product that has been wrongly washed and maintained.

## Maintenance

Every time you use Protac SensCircle® check for flaws and defects. Once a year inspect the pad's seams, zip, fabric, filling and general condition. If you become aware of a serious flaw or defect, stop using your Protac SensCircle® until the flaw has been remedied and safety restored.

## Transport and storage

Protac SensCircle® comes ready for use

For transport, the Protac SensCircle® is packed in a carrying bag and plastic sack to protect the pad against dirt and moisture.

When not using your Protac SensCircle®, you should store it carefully in the carrying bag to protect the pad against dirt and moisture.

GB

## Disposal

When disposing of a Protac SensCircle®, it can be divided into the following individual parts: Plastic and fabric. All parts can be disposed of as combustible waste. Waste should be disposed of in accordance with applicable regulations.

## Warranty

There is a 2-year warranty for manufacturing defects

### The warranty is void in the following instances:

- Overload of the Protac SensCircle®
- Heavy-handed use of the Protac SensCircle®
- Lack of, or improper maintenance of the Protac SensCircle®

**Traceability:** the month and year of manufacture are marked on the back of the care label.

## Key to symbols

Example only. Always follow the recommended care instructions on the label found inside the product

**Washing bowl:** Maximum washing temperature (see care label)

**Warning:**  
See under **Safety**

**Triangle:**  
Do not bleach

User information

**Square with a circle:**  
tumble drying possible (see care label)

For indoor use

**Iron:**  
Do not iron

Manufacturer

**Circle:**  
Do not dry clean

Product serial number (see care label)

## Technical specifications

### Dimensions and weight:

Length/height:	2.50/0.35 m
Total weight – Oxford-3800 model with plastic balls and granules (polystyrene):	approx. 4.5 kg.
Total weight – Mess-s2600 model with granules (polystyrene):	approx. 3.5 kg.
Weight of outer cover:	approx. 1.0 kg.
Weight per module with plastic balls:	approx. 2.0 kg.
Weight per module with granules (polystyrene):	approx. 0.8 kg.

**Standards:** Comply with general requirements for aids acc. to the following harmonized standards:

**Clinical evaluation:** EN 12182:2012

**Risk analysis:** DS/EN ISO 14971

**Fire test:** Tested in accordance with the standards stipulated in the table

GB

Fire test	EN 1021-1:2014	EN 1021-2:2014	BS5852-1:1979	BS5852-2:1982
Fabric Oxford-3820 & EVA plastic balls	X (cigarette test)	X (match flame test)		
Fabric Mess-2600 & granules (polystyrene)			X (cigarette test)	
Granules (polystyrene)				X (flame test)

The Protac SensCircle® contains 3 modules, filled with either EVA plastic balls or granules (polystyrene).

### Service information

Do not use balls in your Protac SensCircle® other than those used by the manufacturer.

### Guide

- to daily inspection of the your Protac SensCircle® before use

No.	Check	Remarks
1	<ul style="list-style-type: none"><li>All parts are in place, no loose parts (modules)</li></ul>	
2	<ul style="list-style-type: none"><li>Stitching ok</li></ul>	

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### Protac A/S